



Flamirins®

Isotonic buffered wound cleanser

Indications

Flamirins® is used to clean wounds and to loosen adhered dressings.
Clean wounds heal better.

Flamirins® mechanically removes foreign bodies and dead tissue from the wound, thus reducing the risk of infection.

Properties

Flamirins® is a solution with the following properties:

- buffered (acid pH)
- isotonic

These properties ensure that any small wound can be cleaned simply, quickly and thoroughly.
Once opened, Flamirins® can be reused several times until the indicated expiry date.

Instructions for use - 250ml

1. Turn the nozzle to the “SPRAY” or “STREAM” position by simultaneously pushing up the tab under the nozzle.
2. Spray the solution in the wound.
3. Dab the wound dry carefully
4. Put the nozzle in “STOP” position.

Precautions

- For external use only.
- Keep out of the reach of children.
- Do not use in case of irritation or if allergic to one of the ingredients.
- Do not use as an antiseptic solution.
- Do not use on the eyelids or in the eyes. If Flamirins[®] does come into contact with the eyes, rinse for at least 5 minutes with running water.
- Contact a doctor if the wound becomes worse.

Storage

Store at room temperature (< 25°C) in a dry place and in the original container.
The expiry date (USE BY: year/month) is the first day of the stated month in that year.
Close the bottle properly immediately after use.

Ingredients

Purified water, sodium chloride, sodium phosphate, polysorbate, citric acid, propylene glycol, methyl paraben, propyl paraben.

Presentations

- 250ml

Flamirins[®] is a medical device.