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Distributors / Authorized representative

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YE: Alsbay for drugs and medical appliances - Republic of Yemen.

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Flaminal® Hydro

This medication

- This medication is a product that affects your health, and its consumption contrary to the instructions poses a risk to you.
- Strictly follow the doctor's prescription, the specified method of use, and the instructions of the pharmacist who dispensed it to you.
- The doctor and the pharmacist are the experts in the medicine, its benefits and risks.
- Do not interrupt the specified period of treatment on your own.
- Do not repeat the prescription without consulting your doctor.
- Keep all medications out of the reach of children.

The Council of Arab Health Ministers and The Federation of Arab Pharmacists

Properties

Flaminal® Hydro is a new class of wound healing agent, called Enzyme Aginogel®. It consists of an antibacterial enzyme system embedded in hydrated alginates.

Flaminal® Hydro supports the healing of a wound in a number of ways:

1. **Flaminal® Hydro** keeps the wound clean.
2. **Flaminal® Hydro** keeps the wound moist.
3. **Flaminal® Hydro** offers antimicrobial protection.
4. **Flaminal® Hydro** is safe for the skin and wound tissue.
5. **Flaminal® Hydro** reduces wound odour caused by bacteria.
6. **Flaminal® Hydro** helps to reduce excessive protease activity (protease = enzyme that breaks down proteins).
7. **Flaminal® Hydro** reduces bacteria released from a biofilm.
8. **Flaminal® Hydro** protects the wound border.

These are the prime properties for ensuring rapid wound healing.

Flaminal® Hydro is indicated for slightly to moderately exuding wounds such as:

- leg ulcers
- diabetic ulcers
- 2nd degree burns (deep, superficial)
- wounds from radiotherapy
- oncology wounds
- complex grazes
- wounds from dermatosurgery

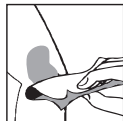
How to use

Applying or renewing **Flaminal® Hydro** is easy and painless. Proceed as follows:

1. First clean and rinse the wound
2. Dab and dry carefully
3. Cover the entire wound bed with a sufficiently thick layer (e.g. 4-5 mm) of **Flaminal® Hydro**
4. Cover **Flaminal® Hydro** with a dressing. The choice of dressing should be based on the type of lesion:
 - Dry wound: transparent film (polyurethane) or paraffin gauze fixed by a non-adhesive bandage
 - Slightly exuding: non-adherent dressing fixed by a non-adhesive bandage or by a hypoallergenic, adhesive tape
 - Moderately exuding: absorbent, non-adherent compress fixed by a non-adhesive bandage or a hypoallergenic, adhesive tape

clean

dry

apply

cover

Good to know

- Check the dressing daily. If it is leaking or if there is insufficient gel, **Flaminal® Hydro** should be changed and a new dressing applied.
- During **Flaminal® Hydro** treatment, dry whitish alginate flakes may appear on the wound border. They should not be removed. The flakes will prevent wound border maceration. Maceration slows the healing process of the wound.
- If dry whitish alginate flakes appear in the wound, cover **Flaminal® Hydro** as you would with a dry wound. The flakes in the wound will disappear when wound moisture balance is restored.
- Should **Flaminal® Hydro** liquefy too quickly, the wound is too exuding for **Flaminal® Hydro**. Use **Flaminal® Forte**.
- The dressing with **Flaminal® Hydro** can remain in place as long as the gel structure is intact (1 to 4 days, depending on the amount of exudate).

- In the initial phase of the treatment the wound may seem to become larger. This is a normal part of the healing process in which dead tissue is eliminated.
- The colour of the alginates may change slightly due to their natural origin. Colour changes have no effect on the quality of **Flaminal® Hydro**.

Special wounds

- Leg ulcers: Adhesives often cause allergic reactions, particularly in patients who suffer from leg ulcers. Use a non-adhesive bandage.
- Undermined wounds: Put sterile gauze tampons in the cavities as a support and apply a sufficient amount of **Flaminal® Hydro** in the cavities between the tampons.
- Fistulas and small deep cavities: Inject **Flaminal® Hydro** carefully into the cavity using a dedicated cannula.
- Exuding wounds: Apply a thicker layer of **Flaminal® Hydro** or use **Flaminal® Forte**.
- Heel ulcers: Apply **Flaminal® Hydro** and cover with a V-shaped gauze compress.

Precautions

Some wounds may indicate a complementary treatment. Consult a physician or wound specialist.

Flaminal® Hydro can be used on infected wounds under medical supervision. Alginates have a faint odour. This odour has no influence on the healing process. Should there be a sudden distinct odour, consult a physician or wound specialist.

Flaminal® Hydro is not indicated in patients with a known allergy to one of the components (see under "Composition").

Flaminal® Hydro cannot be used on the eyelids or in the eye. Should it come into contact with an eye, rinse the eye thoroughly with running water and consult a physician.

Storage

Keep at room temperature (below 25°C) in a dry place in the original pack. The expiry date (USE BY) Year/Month is the first day of the indicated month and year.

Recap the tube or jar carefully immediately after use.

Once opened, and if recapped carefully, a tube of **Flaminal® Hydro** can be stored and used until the expiry date on the tube and box. A jar of **Flaminal® Hydro** 500g however cannot be used more than one week after its first opening.

We recommend that a pack of **Flaminal® Hydro** be used for one patient only (single patient).

Reporting of side effects

- Saudi Arabia:

The National Pharmacovigilance and Drug Safety Centre (NPC)
Fax: +966-11-205-7662
Call NPC at +966-11-2038222, Exts: 2317-2356-2353-2354-2334-2340.
Toll free phone: 8002490000
E-mail: npc.drug@sfgda.gov.sa
Website: www.sfgda.gov.sa/npc

- Other GCC States:

Please contact the relevant competent authority.

Presentations

- Tube of 10g
- Tube of 15g
- Tube of 25g
- Tube of 30g
- Tube of 40g
- Tube of 50g
- Box of 5 tubes of 15g
- Jar of 500g

Not all presentations are available in every country.

Composition

Alginate (total alginate content 3.5% (g/g)); macrogol; hydroxypropylcellulose; enzyme system (glucose oxidase, lactoperoxidase, glucose, guaiacol); potassium sorbate; potassium iodide; buffer; purified water Ph.Eur.

Legal manufacturer

Flen Pharma NV/SA, Blauwestenstraat 87, B-2550 Kontich, Belgium
Manufactured by Conforma NV/SA, Zenderstraat 10, B-9070 Destelbergen, Belgium
for Flen Pharma NV/SA MADE IN BELGIUM

CE 0344

Medical Device
Date of issue: 2016/02

0904-649 /C
I111v2